

WHAT IS CLEAN AND CLEAR?

Clean and Clear™ is a liquid product balanced by a combination of enzymes and surfactants. It has a number of unusual capabilities:

1. It will eliminate organic matter and odor in evacuation systems.
2. It is non-toxic and biodegradable.
3. It is harmless to plastic, rubber and plumbing.
4. It is non-foaming.
5. It cleans dental and surgical instruments.
6. It removes stains and soil from garments, fabrics and carpets.

NEW!
& IMPROVED

HOW DOES THE ACTION OCCUR?

The elimination of organic matter from evacuation systems is accomplished by way of a combination of mechanical and chemical processes with the use of Clean and Clear™.

Enzymatically active compounds will hydrolyze proteins, and solubilize fatty acids and starch. The enzymes in Clean and Clear™ surfactants show them to be 90% biodegradable in as little as seven days.

Do not use Clean and Clear™ with caustics or disinfectants as they inhibit enzymes.

CLEAN AND CLEAR USES

EVACUATION AND COMMERCIAL DRAIN SYSTEM CLEANING:

At the end of the day, add 3 ounces of Clean and Clear™ to 1 gallon of water in suitable container or sink. With system full open, draw solution through evacuator hose and saliva ejectors. Clean and Clear™ effectively removes organic debris and blood, leaving instruments clean and free of any build-up or residues. Extra heavy or old dried deposits may require extra soaking time or light brushing.

SURGICAL DENTAL AND MEDICAL INSTRUMENT CLEANING:

Dilute Clean and Clear™ 1 part to 10 parts water. Soak instruments for 5 to 10 minutes, then rinse with water. Clean and Clear™ effectively removes organic debris and blood, leaving instruments clean and free of any build-up or residues. Extra heavy or old dried deposits may require extra soaking time or light brushing.

STAIN REMOVAL IN GARMENTS, FABRIC AND CARPETS:

Wet stain with 10% solution as used in cleaning instruments. Let stand 3 to 5 minutes, then blot out or rinse out. For stubborn stains, repeat as necessary. Always use mixed amount — do not store mixed solution.

SOUTHLAND DISTRIBUTION & SALES

1-800-880-0240



NEW!
LIVE ENZYME

CLEAN and CLEAR™

CLEAN AND CLEAR™

A multi live enzyme and surfactant solution designed to clean and eliminate odors in vacuum systems by dissolving organic matter.

- Non-foaming
- Non-toxic
- Non-caustic
- Biodegradable
- Harmless to plumbing, plastic and rubber
- Compliant with EPA guidelines for water discharge
- PH safe for all EPA mandated amalgam and mercury separators

CLEAN AND CLEAR™

- A non-foaming solution specifically designed for safe use in all evacuation systems.
- Excellent for cleaning all organic stains and odors from surgical, dental, medical instruments, garments, fabrics and carpets.
- Easy and economical to use. Dilute 3 ounces Clean and Clear™ to 1 gallon warm water for evacuation and commercial drain systems. (170 applications)

DILUTE 1 part Clean and Clear™ to 10 parts warm water for instruments and fabric.

AVAILABLE in gallon bottles.



SOUTHLAND DISTRIBUTION & SALES • 1-800-880-0240 | SDSSOUTHLAND.COM